


SOL-GUARD™ Safety Needle

Reduction of Needle-stick Injury


Compatible with luer slip and luer lock syringes

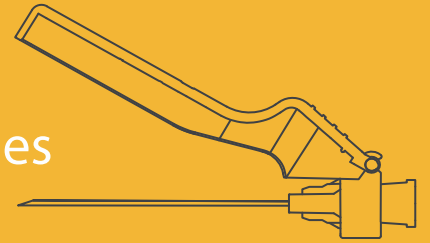


How to Use

- 1 Connect the syringe with blunt fill/filter needle for drug preparation.
- 2 Draw up the medication following facility protocol.
- 3 Replace blunt fill needle by SOL-GUARD™ Safety Needle.
- 4 Open the safety shield fully towards the syringe barrel to be able to observe the needle and injection site during injection procedure.
- 5 Inject the medication according to facility protocol.
- 6 Withdraw the needle and activate the safety mechanism immediately.
- 7 Discard activated needle into sharps container according to facility protocol.

How to Activate Safety Mechanism

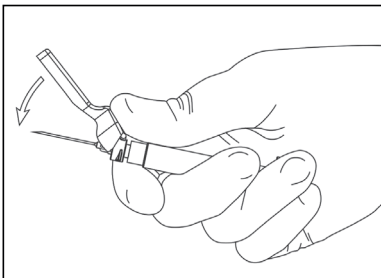

Use your thumb to push the safety shield over the needle until an audible “CLICK” is perceived. This indicates successful activation of the safety mechanism.

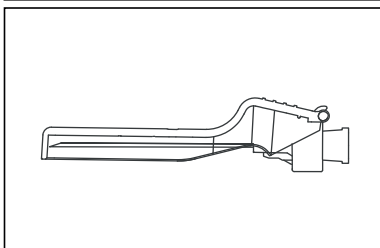

Safety mechanism properly activated.



For more information, please contact:

Sol-Millennium Europe Sp. z o.o.
Twarda 18, PL-00105 Warsaw, Poland
Tel: +48 22 202 67 23,
e-mail: info.EU@sol-m.com
sol-m.com

EMA-SIN-169-0121



SOL-MILLENNIUM®
Building a Healthier Tomorrow